

derma+flex[®] QS[™]

high viscosity tissue adhesive

DIRECTIONS FOR USE

STEP 1:

The incision or trauma site must be clean and dry before applying adhesive. Ensure that the wound edges are easily approximated.

STEP 2:

Screw the Applicator Tip onto the threaded tube in a clockwise direction until the applicator is tight and seated on the tube. The applicator will puncture the foil membrane on the tube once seated and adhesive will be allowed to flow.

STEP 3:

While holding the tube upright and away from patient, squeeze the tube from the bottom moving the adhesive upward until it becomes visible on the applicator surface and is flowing freely.

STEP 4:

While approximating skin edges, apply adhesive over the entire wound site. Move the flat surface applicator in a back and forth motion until adhesive has created a smooth and even coat covering the entire wound site.

STEP 5:

Continue to hold the wound edges in approximation until the adhesive becomes tacky. Typically less than 20 seconds.

STEP 6:

Once adhesive has completely polymerized (non-tacky) you may cover the site with a secondary bandage. If a secondary bandage is used, DO NOT apply to the adhesive area until it is completely tack free. Applying while tacky may result in adhesive removal when removing secondary bandage and possibly dehiscence of the wound site.

STEP 7:

Discard adhesive device according to normal protocol after use.

